

Super Absorbent Dressing (Type II) Instruction for Use

Products description

Type II of Super Absorbent Dressing is composed of 3 layer structure, which is composed of two layers of PP non-woven fabric combined with a thick superabsorbent fiber core. Two layers of PP non-woven fabric can be used as wound contact layer or backing layer without distinction, and can be used at will; at the same time, the material is soft in contact with the wound surface, improving patient comfort. The superabsorbent core has super absorbent capacity, can absorb several times more liquid than its own weight, and will not release retained exudates even under mechanical pressure. Therefore, the superabsorbent dressing (Type II: Thick type and three layers structure) has a very high absorption capacity, which helps to reduce dressing changes and promote wound healing.

Intended purpose

Super Absorbent Dressing is a highly absorbent wound dressing for the management of moderate to highly exuding wounds. It was designed to absorb wound exudate and provide a moist wound healing environment to promote wound healing.

Indications

It is indicated for acute and chronic wounds with moderate to heavily exudate, such as, diabetic foot ulcers, pressure ulcers, leg venous and arterial ulcers, first and second degree burns, postoperative wounds, and traumatic wounds.

Intended user:

The product shall be used by or under the supervision of a healthcare professional.

Function principle

Super absorbent dressing (Type II: Thick type and three layers structure), this dressing has good exudate management ability. It absorbs several times the exudate and locks it through the function of the super absorbent core layer, thereby maintaining the moist environment of the wound, reducing the number of dressing changes, and promoting The role of wound healing. The front and back sides of the dressing are soft materials that can touch the wound and do not stick to the wound surface.

Intended patient population

Patient who has the indications:

Acute and chronic wounds with moderate to heavily exudate, such as, diabetic foot ulcers, pressure ulcers, leg venous and arterial ulcers, first and second degree burns, postoperative wounds, and traumatic wounds.

Clinical benefit:

- For wound management: Reduce the frequency of dressing changes.

Contraindications:

- 1.The Super Absorbent Dressing cannot be used on dry wounds,heavy bleeding wounds, third degree burns and surgical implantation.
- 2.Check the wound for signs of infection before use,if infection occurs,see a health care professional.
- 3.Do not use on the patients with a known hypersensitivity to the product itself or to its components.
- 4.Do not use in tunnelling wound as the product can expand considerably when wound exudate is absorbed.

Complications or side-effects

The human bodies are well tolerated to Super Absorbent Dressing products, however, after combined the available clinical data and post-marketing data of competitive device and benchmark devices,the following situations are identified and they are very rare:

-- Pain on dressing removal.

Instruction for Use**First step:**

Select a size of dressing enough to overlap the wound edges by at least 2-3cm.

Second step:

Gently apply the dressing directly on the wound site with the white hydrophilic side of dressing onto the wound surface.

Third step:

The dressing shall be fixed by an appropriate secondary dressing , such as Adhesive tape, elastic bandage or pressure bandage.

Note:

- 1.The dressing can be left in place for up to 7days, but wearing time should be depended on level of exudate, because of excellent fluid handling capability of the dressing, it may produce pressure and require change.
- 2.The wound dressing may not be cut to size.

Precautious/Warnings

- 1)Do not reuse. Reusing will cause cross-contamination.
- 2)Do not use if package is damaged or open.
- 3)Do not resterilize.
- 4)Check the wound for signs of infection before use, if infection occurs, see a health care professional.

Storage and transport conditions

The product should be stored under normal room condition, keep away from sunlight and rain. The transportation should prevent the weight, sunlight and rain.

Disposal

Disposal should be handled according to local environment procedures.

Other information

1. The intended user does not need training for this product.
2. If any serious incident has occurred in relation to the use of this dressing, it should be reported to Winner Medical Co., Ltd.



Winner Medical Co., Ltd.
 Winner Industrial Park, No.660 Bulong Road, Longhua District, 518109
 Shenzhen, China



Shanghai International Holding Corp. GmbH (Europe)
 Eiffestraße 80, 20537 Hamburg, Germany



Symbol					
Meaning	Batch code	Use-by date	Do not re-use	Manufacturer	Date of manufacture
Symbol					
Meaning	Sterilized using Ethylene Oxide	Keep dry	Medical device	Keep away from sunlight	Authorized representative
Symbol					
Meaning	Unique device identifier	Importer	This way up	Single sterile barrier system	Do not use it if the package is damaged
Symbol					
Meaning	CE marking		Consult instructions for use or consult electronic instructions for use		Catalogue number

Effective Date: 2025.05.26

Version: A/4