

Foam dressing without border

Instructions for Use

Product description

Foam dressing without border is a highly absorbent dressing. It consists of a polyurethane foam and a waterproof backing film with high permeability to allow for an effective exudate management of moderate to highly exuding wounds, and provides a moist wound healing environment to promote wound healing.

Intended purpose

Foam dressing without border is a wound dressing that absorbs wound exudate and provides a moist wound healing environment to promote wound healing.

Intended user

The product shall be used by or under the supervision of a healthcare professional.

Function principle

Through an excellent exudate management, foam dressing without border creates a moist wound environment for optimal wound healing to take place.

Indications

Foam dressing without border is designed for the management of moderate to heavily exuding wounds such as leg ulcers and pressure sores. Throat foam dressing without border mainly for exudate absorption at the tracheostomy site.

Intended patient population

Patient who has the indications: moderate to heavily exuding wounds such as leg ulcers and pressure sores. Throat foam dressing without border mainly for exudate absorption at the tracheostomy site.

Contraindications

1. Do not use product in the event of symptomatic fungal infection.
2. Do not use on the patients with a known hypersensitivity to the product itself or to its components.

Adverse reaction

Very occasionally, there will be some potential risks associated with the use of foam dressing with the following:

- Adverse reactions (eczema, edema, erythema, erosion etc.)
- Wound infections

- Risk of maceration (fluid leakage)
- Risk of skin peel

Instructions for use**First step:**

Clean the wound area, hands and instruments in accordance with normal procedures. Ensure the peri-wound skin is dry.

Second step:

Select a size of foam dressing without border enough to overlap the wound edges by at least 2 - 3 cm. If necessary, cut to shape.

Third step:

Gently apply the wound contact side which without polyurethane film directly on the wound site.

Fourth step:

Foam dressing without border may be secured by any secondary the clinician prefers roll gauze, elastic or cohesive wraps, film dressing, tubular soft cloth surgical tape. These dressings should be large enough to cover the foam dressing.

Note:

The dressing change interval may be several days. Change the dressing before it is fully saturated, at signs of leakage, or as indicated by clinical practice. The same product cannot be used for more than 7 days, continuous use can not be more than 30 days.

Warnings

- 1) Do not reuse. Reusing will cause cross-contamination.
- 2) Do not use if package is damaged or opened.
- 3) Do not use it together with oxidizing agents such as hypochlorite solutions or hydrogen peroxide.
- 4) For non-sterilized products, sterilize before use.

Storage and transport conditions

1. Foam dressing without border should be stored in warehouse under normal room condition, temperature and humidity of room will not be specified strictly but require cool and dry. Direct sunlight shall be avoided.

- The transportation should be to prevent the weight, sun and rain and rain Dripping.

Disposal

Disposal should be handled according to local environment procedures.

Other informations

- Patients can undergo an MRI scan while using this dressing.
- If have any serious incident that has occurred in relation to the device, please reported to the manufacturer and the competent authority



Winner Medical Co., Ltd.

Winner Industrial Park, No. 660 Bulong Road, Longhua New District, Shenzhen, China
















Shanghai International Holding Corp. GmbH (Europe)

Eiffestraße 80, 20537 Hamburg, Germany



The meaning of the symbols used:

Symbol	Meaning
	Manufacturer
	CE marking
	Authorised representative
	Date of manufacture
	Use-by date

	<p>Batch code</p>
	<p>Catalogue number</p>
	<p>Medical device</p>
	<p>Sterilized using irradiation</p>
	<p>Do not use if package is damaged</p>
	<p>This side up</p>
	<p>Keep away from sunlight</p>
	<p>Keep dry</p>
	<p>Do not re-use</p>
	<p>Do not re-sterilize</p>
	<p>Consult instructions for use</p>
	<p>Caution</p>
	<p>Not contain natural rubber latex</p>

Effective Date: 2022.07.29

Version: A/1