

Hydrocolloid Dressing Thin

Instruction for Use

Products description

Hydrocolloid Dressing Thin is a thin and flexible dressing that makes it perfectly easy to dress any part of the body. It is made of three parts: hydrocolloid, polyurethane film and release paper. Hydrocolloid layer is composed of sodium carboxymethyl cellulose (CMC) and hot melt adhesive, it has the function of absorbing liquid and allows the dressing to adhere stably to the wound and surrounding skin. The polyurethane film is moisture vapor permeable, waterproof and impermeable to liquids, bacteria. It can be used for management of non to low exuding wounds, including donor sites, postoperative wounds and skin abrasions. This product can provide a moist healing environment for the wound.

Intended purpose

Hydrocolloid Dressing Thin is intended for management of non- to low-exuding wounds that the injuries to skin have breached epidermis or dermis, such as donor sites, postoperative wounds and skin abrasions. This dressing is for short-term use (single use time ≤ 7 days and accumulated use time ≤ 28 days). This is a professional use only and single use device.

Indications

Hydrocolloid Dressing Thin is indicated for a wide range of non- to low-exuding acute wounds, such as donor sites, postoperative wounds, skin abrasions.

Intended user

The product shall be used by or under the supervision of a healthcare professional.

Function principle

In the presence of wound exudate, the hydrocolloid absorbs liquid and forms a gel that

does not adhere to the wound. The polyurethane film is moisture vapor permeable, waterproof, and impermeable to liquids, bacteria, which helps to provide a moist healing environment for the wound.

Intended patient population

Patient who has the indications:

Non- to low-exuding acute wounds, such as donor sites, postoperative wounds, skin abrasions.

Clinical benefit

Relieve patient pain during treatment

Contraindications


Do not use on the patients with a known hypersensitivity to the product itself or to its components.

Complications or side-effects

Very occasionally, there will be some potential risks associated with the use of Hydrocolloid Dressing with the following:

- Adverse reactions (Intolerance, erythema, hypergranulation etc.)
- Wound infections
- Risk of maceration (fluid leakage)

Instruction for use

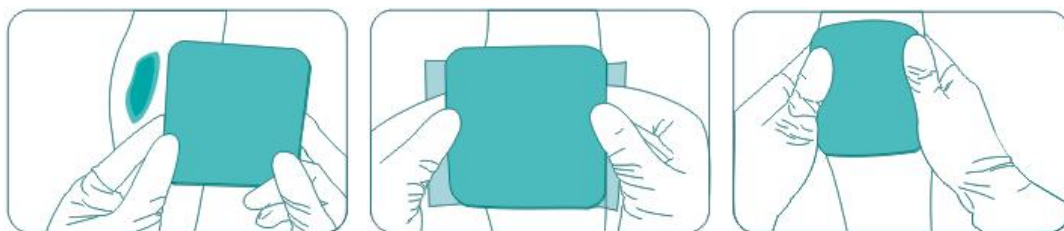

First step:

Clean the wound area in accordance with normal procedures. Ensure the peri-wound skin is dry.

Second step:

Products shall keep clean and sterile before using, do not use if package is damaged. Select a size of hydrocolloid dressing enough to overlap the wound edges by at least 2 - 3 cm. if necessary, cut to shape.

Third step:

Remove the dressing from the pouch, tear off the release film and apply the adherent side to the wound. Remove the remaining liner and smooth dressing down onto skin.

Fourth step:

For dressing remove, press down on the skin outside of the dressing and gently lift an edge of the dressing. Gently loosen the edges until all are free from the skin surface.

1. The dressing change interval may be several days. Change the dressing before it is fully saturated, at signs of leakage, or as indicated by clinical practice. The same product cannot be used for more than 7 days, continuous use cannot be more than 28 days.
2. Frequent dressing changes on wounds with damaged or delicate skin surrounding the wound is not recommended.

Precautions/Warnings

1. Do not reuse. Reusing will cause cross-contamination.
2. Do not use if package is damaged or open.

Storage and transport conditions

The transportation should prevent the weight, sun and rain and rain dripping.

Disposal

Disposal should be handled according to local environment procedures.

Other information

1. The intended user does not need training for this product.
2. If any serious incident has occurred in relation to the use of this dressing, it should be reported to Winner Medical Co., Ltd.



Winner Medical Co., Ltd.

Winner Industrial Park, No. 660 Bulong Road, Longhua District, 518109 Shenzhen, PEOPLE'S
REPUBLIC OF CHINA



Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg, Germany



Symbol					
Meaning	Batch code	Use-by date	Do not reuse	Manufacturer	Date of manufacture
Symbol					
Meaning	Sterilized using irradiation	Medical device	Keep dry	Keep away from sunlight	Authorized representative
Symbol					
Meaning	Unique device identifier	Importer	This way up	Single sterile barrier system	Do not use if package is damaged
Symbol					
Meaning	CE marking		Do not resterilize	Consult instructions for use or consult electronic instructions for use	

Effective Date: 2025.05.30

Version: A/2