

# Hydrocolloid Dressing with Tapered Edges

## Instruction for Use

### Products description

Hydrocolloid Dressing with Tapered Edges is a dressing with a tapered edge and a thicker core, the edge part is thinner than the core part, and it can be adhered to the skin for a long time. It is made of three parts: hydrocolloid, polyurethane film and release paper. Hydrocolloid layer is composed of sodium carboxymethyl cellulose (CMC) and hot melt adhesive, it has the function of absorbing liquid and allows the dressing to adhere stably to the wound and surrounding skin. The polyurethane film is moisture vapor permeable, waterproof, and impermeable to liquids, bacteria. It can be used for management of non to moderately exuding wounds, including pressure ulcers, leg ulcers, donor sites, postoperative wounds and skin abrasions. This product can provide a moist healing environment for the wound.

### Intended purpose

Hydrocolloid Dressing with Tapered Edges is intended for management of non- to moderate-exuding wounds that the injuries to skin have breached epidermis or dermis, such as pressure ulcers, leg ulcers, donor sites, postoperative wounds and skin abrasions. This dressing is for short-term use (single use time  $\leq 7$  days and accumulated use time  $\leq 28$  days). This is a professional use only and single use device.

### Indications

Hydrocolloid Dressing with Tapered Edges is indicated for a wide range of non- to moderate-exuding chronic and acute wounds, such as pressure ulcers, leg ulcers, donor sites, postoperative wounds, skin abrasions.

### Intended user

The product shall be used by or under the supervision of a healthcare professional.

**Function principle**

In the presence of wound exudate, the hydrocolloid absorbs liquid and forms a gel that does not adhere to the wound. The polyurethane film is moisture vapor permeable, waterproof, and impermeable to liquids, bacteria, which helps to provide a moist healing environment for the wound.

**Intended patient population**

Patient who has the indications:

Non- to moderate-exuding chronic and acute wounds, such as pressure ulcers, leg ulcers, donor sites, postoperative wounds, skin abrasions.

**Clinical benefit**

Relieve patient pain during treatment

**Contraindications**

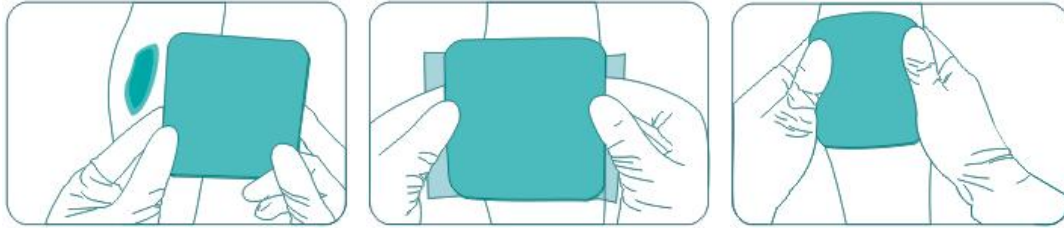
Do not use on the patients with a known hypersensitivity to the product itself or to its components.

**Complications or side-effects**

Very occasionally, there will be some potential risks associated with the use of Hydrocolloid Dressing with the following:

- Adverse reactions (Intolerance, erythema, hypergranulation etc.)
- Wound infections
- Risk of maceration (fluid leakage)

**Instruction for use**

**First step:**

Clean the wound area in accordance with normal procedures. Ensure the peri-wound skin is dry.

**Second step:**

Products shall keep clean and sterile before using, do not use if package is damaged. Select a size of hydrocolloid dressing enough to overlap the wound edges by at least 2 - 3 cm. if necessary, cut to shape.

**Third step:**

Remove the dressing from the pouch, tear off the release film and apply the adherent side to the wound. Remove the remaining liner and smooth dressing down onto skin.

**Fourth step:**

For dressing remove, press down on the skin outside of the dressing and gently lift an edge of the dressing. Gently loosen the edges until all are free from the skin surface.

1. The dressing change interval may be several days. Change the dressing before it is fully saturated, at signs of leakage, or as indicated by clinical practice. The same product cannot be used for more than 7 days, continuous use cannot be more than 28 days.
2. Frequent dressing changes on wounds with damaged or delicate skin surrounding the wound is not recommended.

**Precautions/Warnings**

1. Do not reuse. Reusing will cause cross-contamination.
2. Do not use if package is damaged or open.

**Storage and transport conditions**

The transportation should prevent the weight, sun and rain and rain dripping.

**Disposal**

Disposal should be handled according to local environment procedures.

**Other information**

1. The intended user does not need training for this product.
2. If any serious incident has occurred in relation to the use of this dressing, it should be reported to Winner Medical Co., Ltd.



Winner Medical Co., Ltd.

Winner Industrial Park, No. 660 Bulong Road, Longhua District, 518109 Shenzhen, PEOPLE'S  
REPUBLIC OF CHINA



Shanghai International Holding Corp. GmbH (Europe)  
Eiffestraße 80, 20537 Hamburg, Germany



|                |                              |                |              |                               |                                  |
|----------------|------------------------------|----------------|--------------|-------------------------------|----------------------------------|
| <b>Symbol</b>  |                              |                |              |                               |                                  |
| <b>Meaning</b> | Batch code                   | Use-by date    | Do not reuse | Manufacturer                  | Date of manufacture              |
| <b>Symbol</b>  |                              |                |              |                               |                                  |
| <b>Meaning</b> | Sterilized using irradiation | Medical device | Keep dry     | Keep away from sunlight       | Authorized representative        |
| <b>Symbol</b>  |                              |                |              |                               |                                  |
| <b>Meaning</b> | Unique device identifier     | Importer       | This way up  | Single sterile barrier system | Do not use if package is damaged |

|                |   |   |   |
|----------------|---|---|---|
| <b>Symbol</b>  |  |  |  |
| <b>Meaning</b> | CE marking  | Do not resterilize  | Consult instructions for use or consult electronic instructions for use             |

**Effective Date:** 2025.05.30

**Version:** A/2